

## Nasal Arch

The Nasal Arch can be used effectively to create a nasal on lay where subtle augmentation of the dorsum is required. Care should be taken to place the Nasal Arch appropriately in the dorsum area and to avoid extending the Nasal Arch proximally into the soft nasal cartilage area of the tip. The edge of the Nasal Arch should be feathered to promote a smooth transition from the implant to the patient's natural contour.

Cat#	Description	A	B	C
4244	Small	70mm	13mm	2mm
4245	Medium	70mm	15mm	2mm
4246	Large	70mm	17mm	2mm

